

Sample Submission Form

Please complete one form for each horse being tested.


Name: _____ Male Female

Breed: _____ Color/Markings: _____

Pull - **DO NOT CUT** - hair samples from the mane above the withers. Pull 15-25 hairs by wrapping the hair around your finger close to the skin and pulling quickly and evenly. The DNA is contained in the root bulb (follicle) - it is important that this part of the hair remain intact. Inspect the hairs to see that they have not been broken off before the root. This is particularly important with young foals as their hair tends to be finer and more brittle.


Place the hairs in a straight line across the middle of the form. This area is reserved for samples. Keep the hair together with the roots to the left as indicated. Tape the center of the hair shafts with CLEAR tape to insure the sample stays in place.

SAMPLE SECTION *(Please Do Not Cut Hair)*

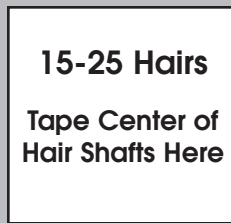

EXAMPLE

root

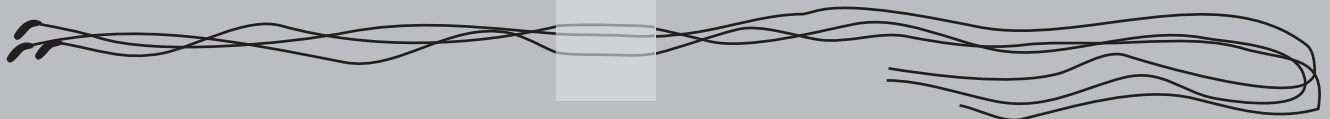

extra length may
be folded over

Fold the hair kit as you would a business letter as indicated by the dotted lines and place in the return envelope provided. Please include your completed **Equine DNA Order Form** along with payment and mail to:

DDC Veterinary
One DDC Way
Fairfield, OH 45014

If you have any questions regarding the hair sample collection procedure,
please call us at **1-800-625-0874**.